



[Previous](#) [Next](#)

Step 6: (7/30/2019 8:35:58 AM) User left click on "Check for new updates (window)" in "Che"

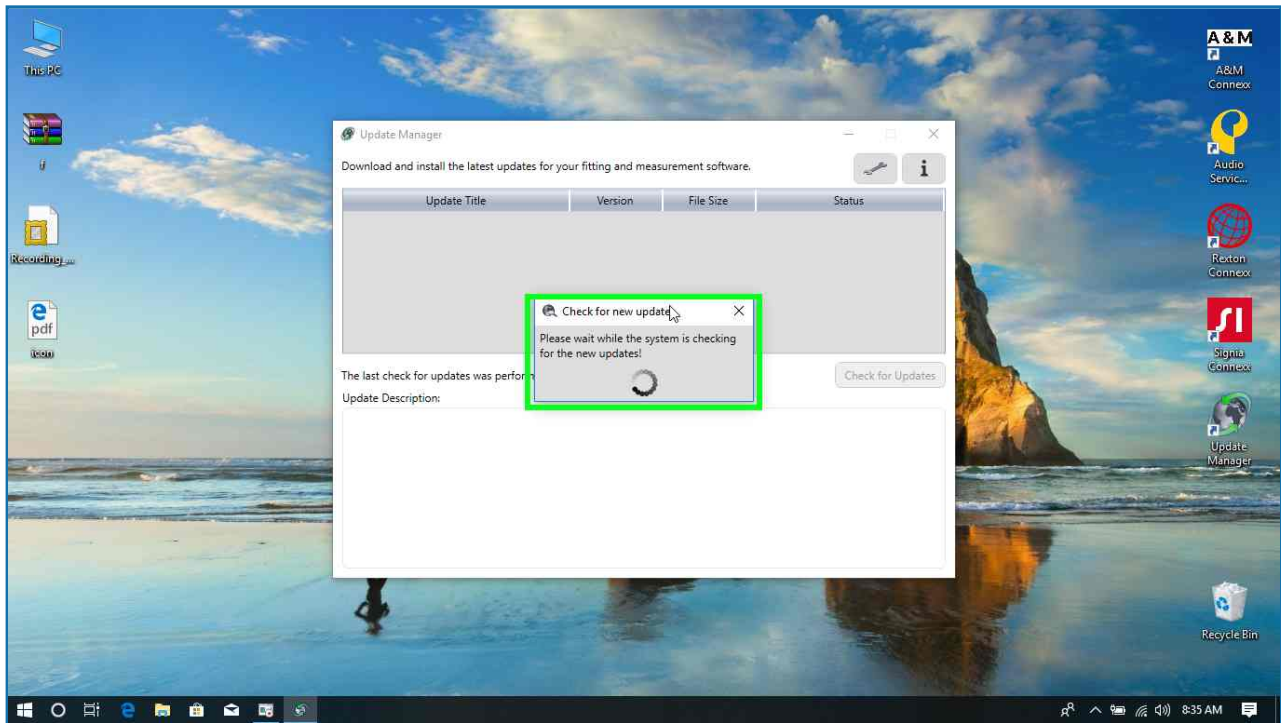


[Previous](#) [Next](#)

Step 7: (7/30/2019 8:36:02 AM) User left click on "Update Manager (window)" in "Upd"

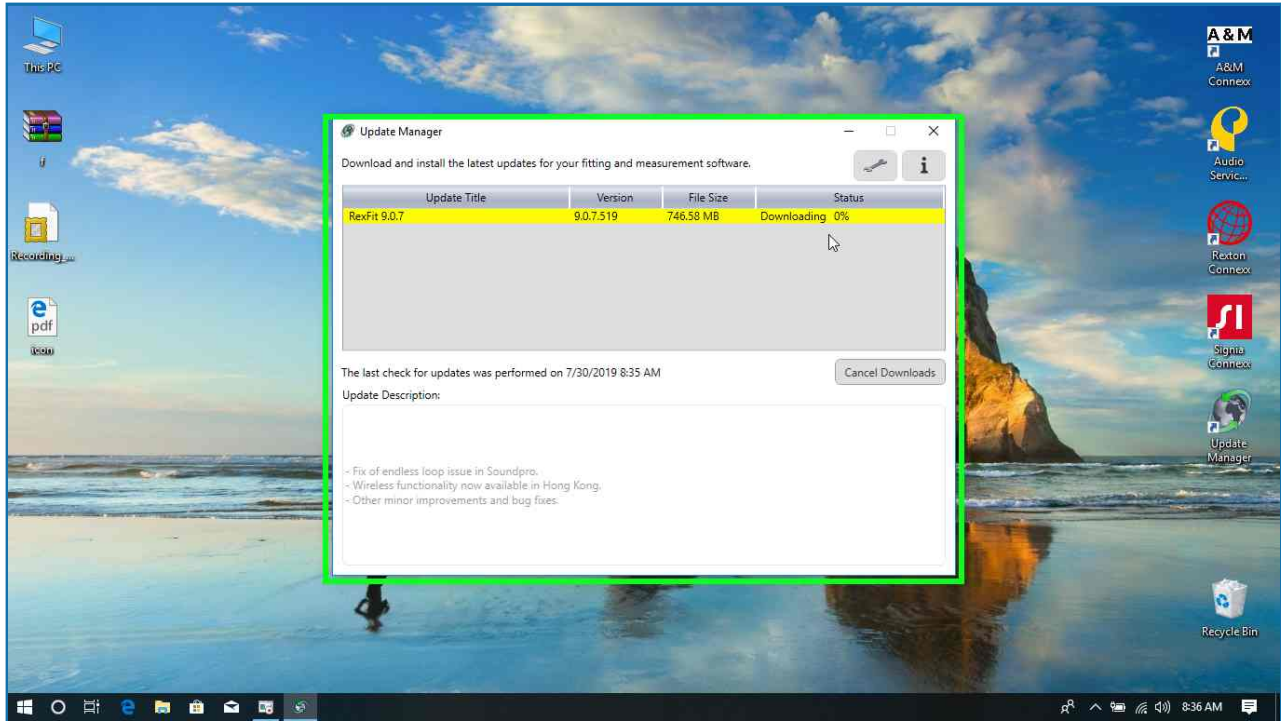


[Previous](#) [Next](#)


Step 8: (7/30/2019 8:36:32 AM) User left click on "Update Manager (window)" in "Upd"

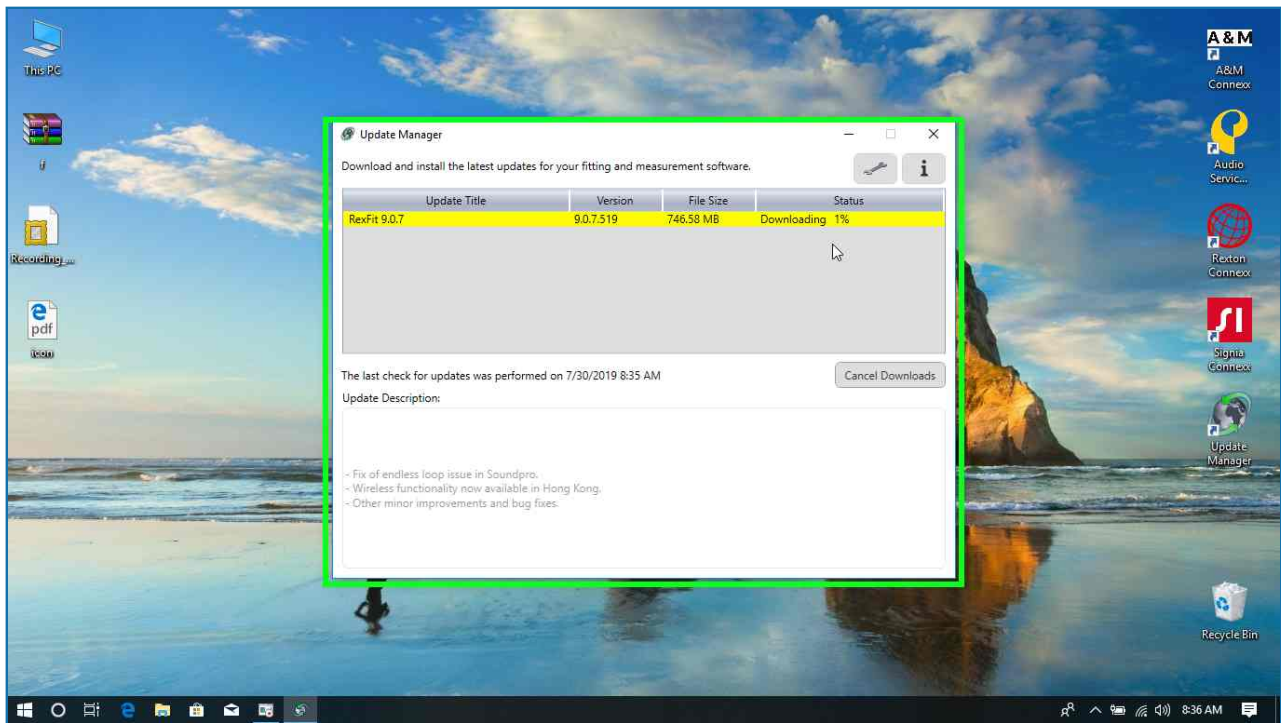

Step 9: (7/30/2019 8:49:06 AM) User left click on "SHS.SAT.UpdateManager.FrontEnd.ViewModels.PackageViewModel (dataitem)" in "Upd"

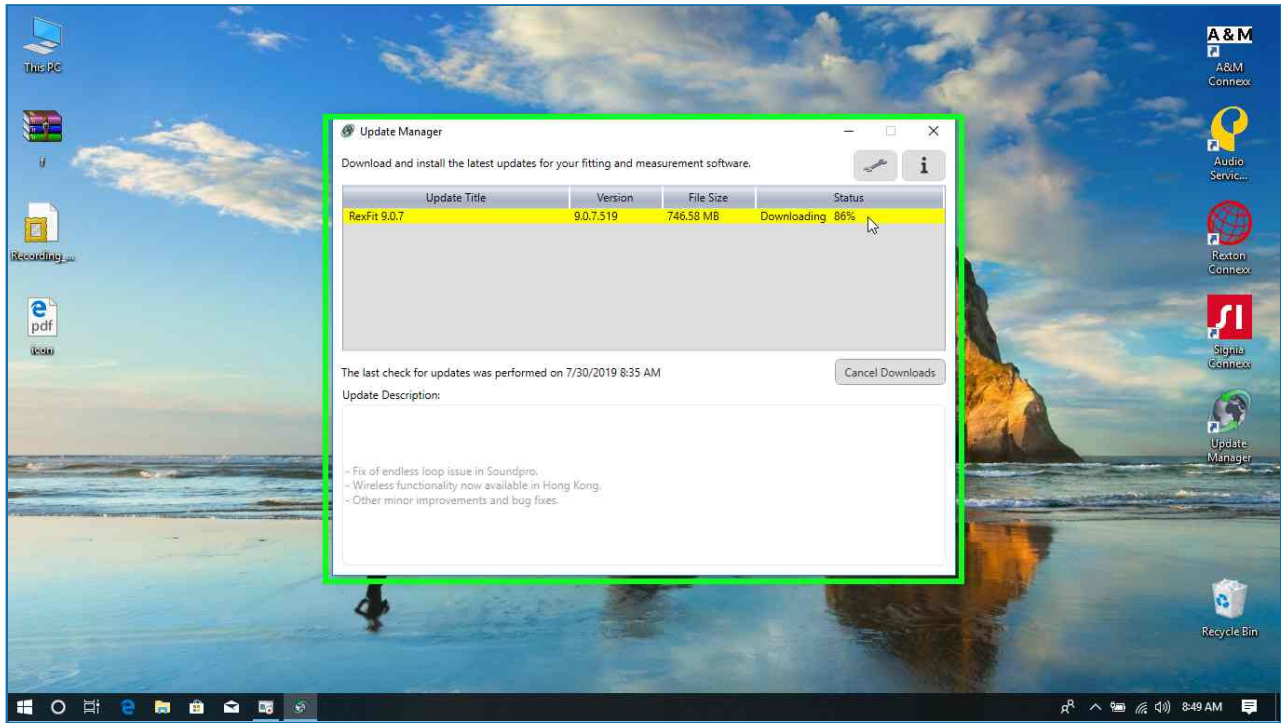


Step 10: (7/30/2019 8:51:05 AM) User left click on "SHS.SAT.UpdateManager.FrontEnd.ViewModels.PackageViewModel (dataitem)" in "Upd"

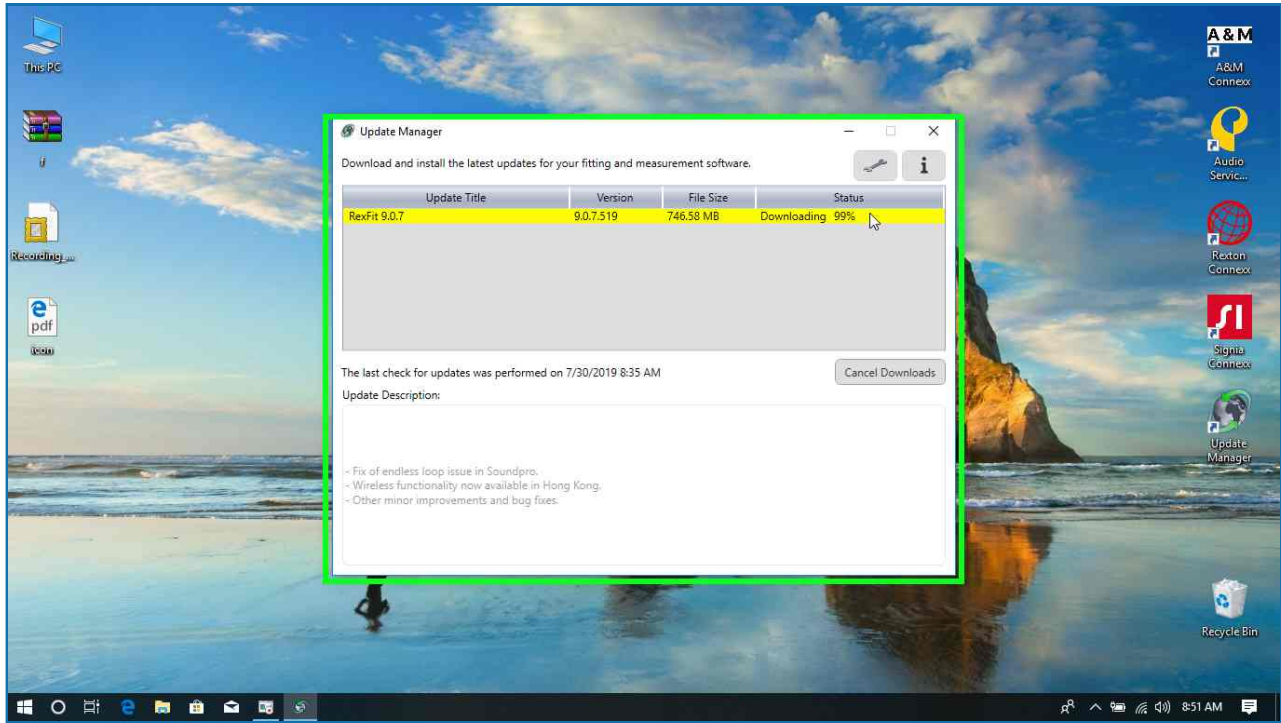


[Previous](#) [Next](#)

Step 11: (7/30/2019 8:51:18 AM) User left click on "Updates have been downloaded. (window)" in "Upd"

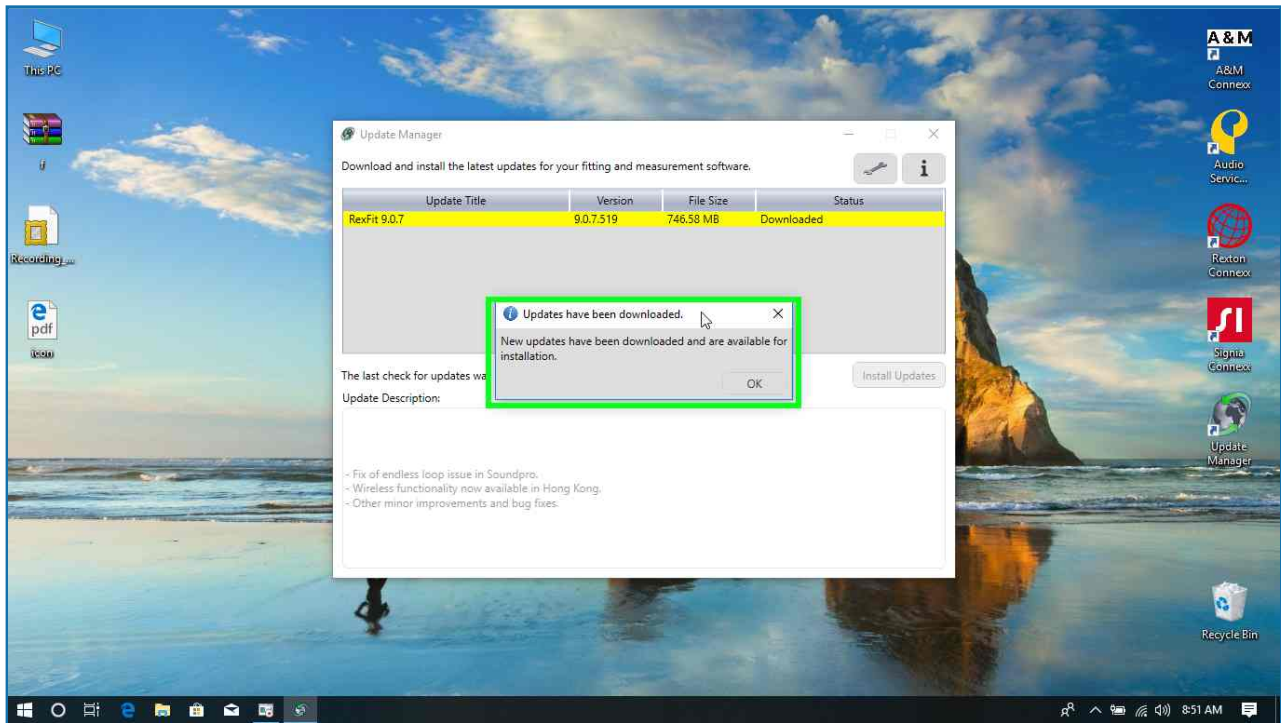


[Previous](#) [Next](#)

Step 12: (7/30/2019 8:51:21 AM) User left click on "OK (text)" in "Upd"

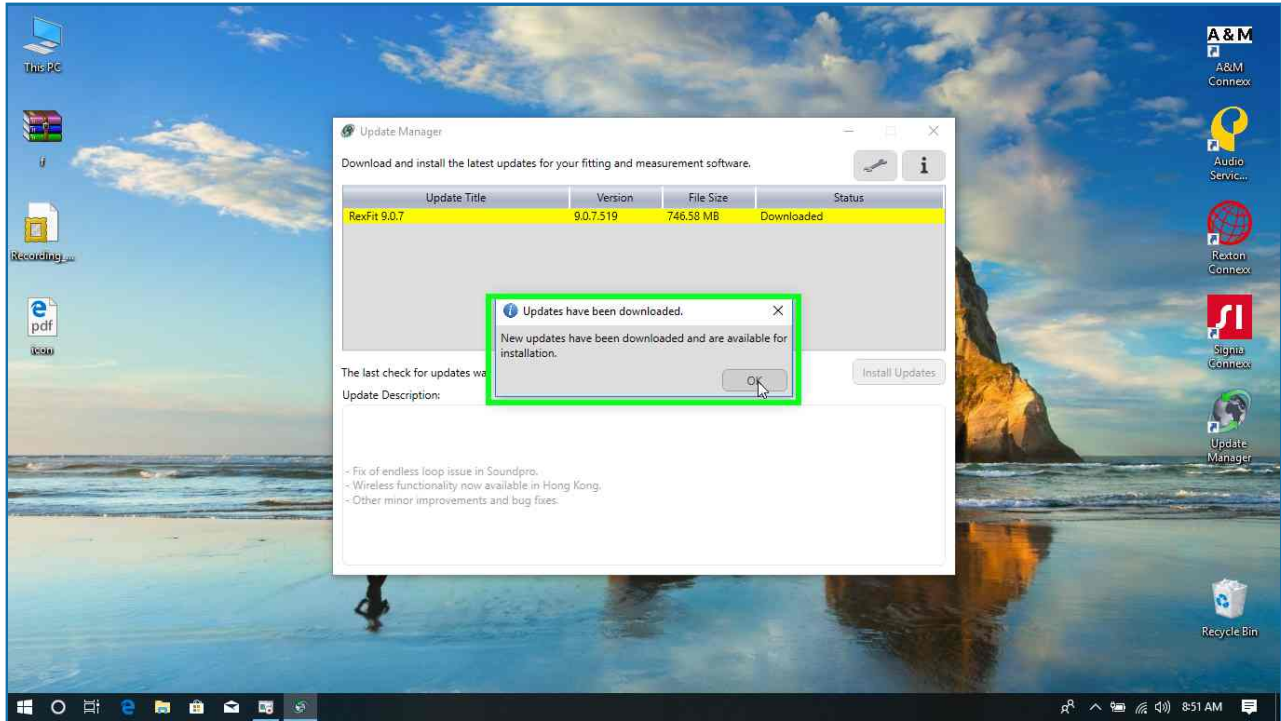


[Previous](#) [Next](#)


Step 13: (7/30/2019 8:51:23 AM) User left click on "Install Updates (text)" in "Upd"

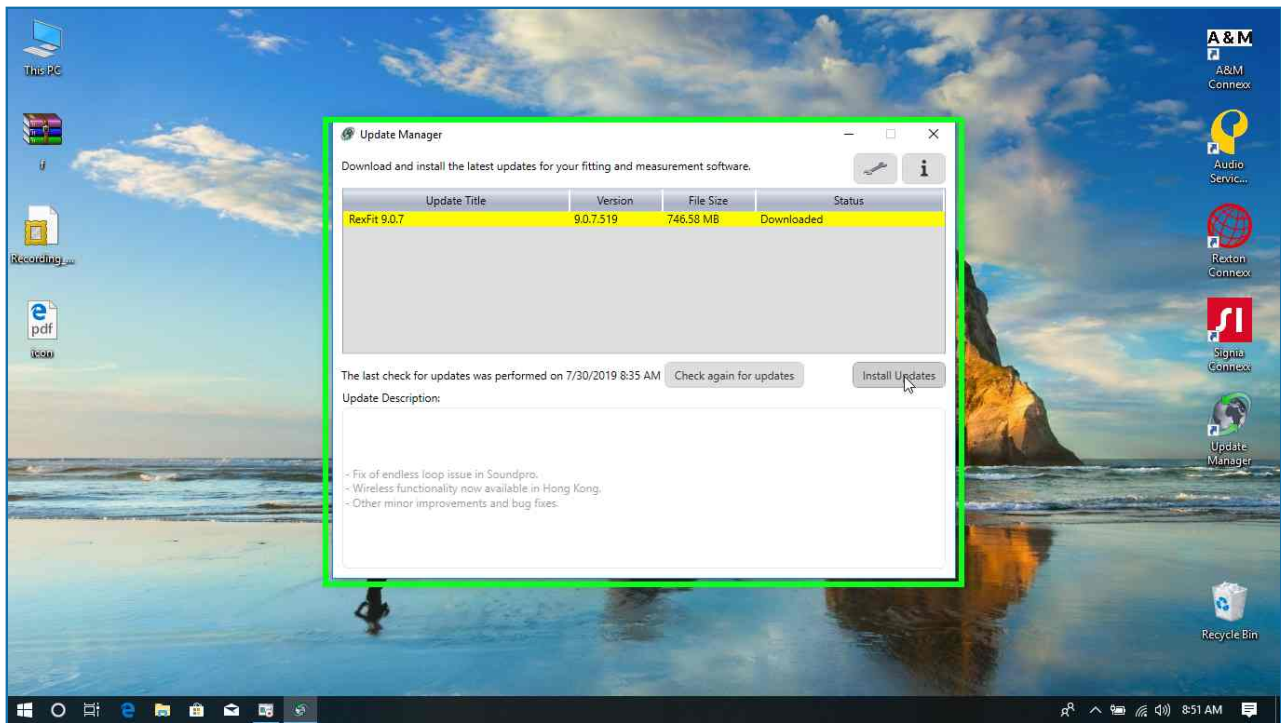


Step 14: (7/30/2019 8:51:30 AM) User left click on "46% Extracting (dialog)" in "46%"

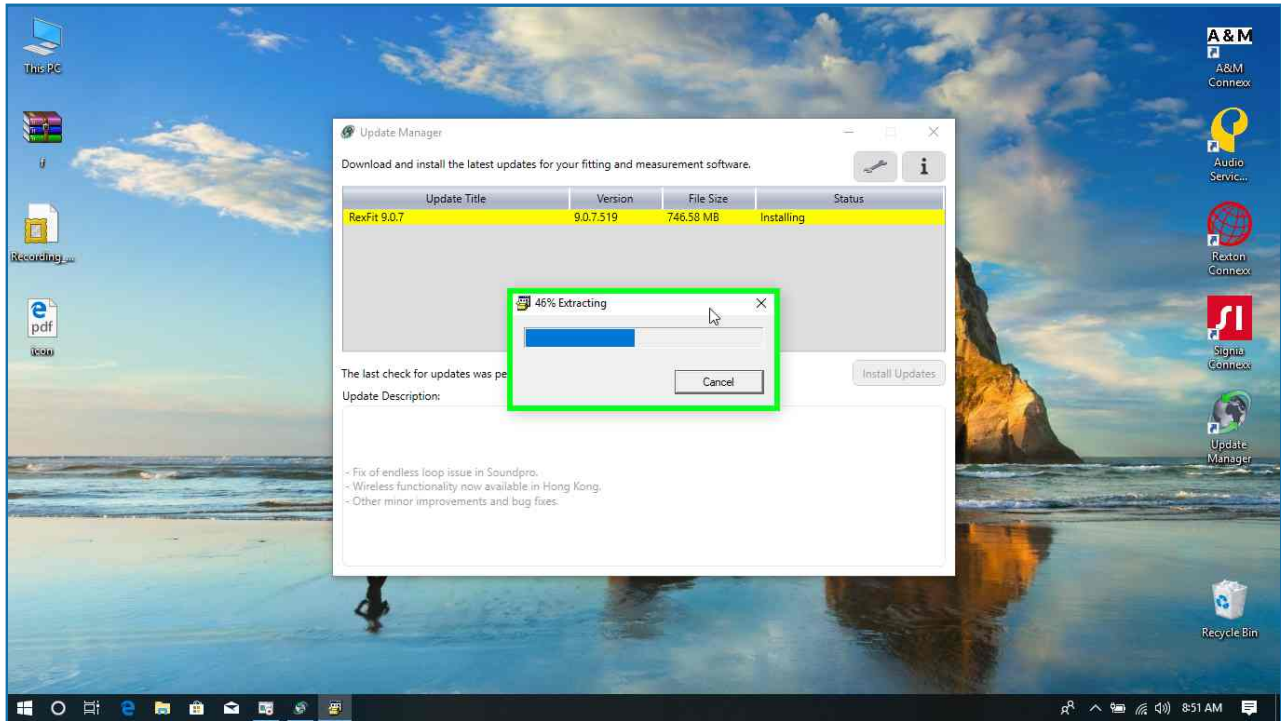


Step 15: (7/30/2019 8:51:51 AM) User left click on "Rexton Fitting Framework (window)" in "Rex"

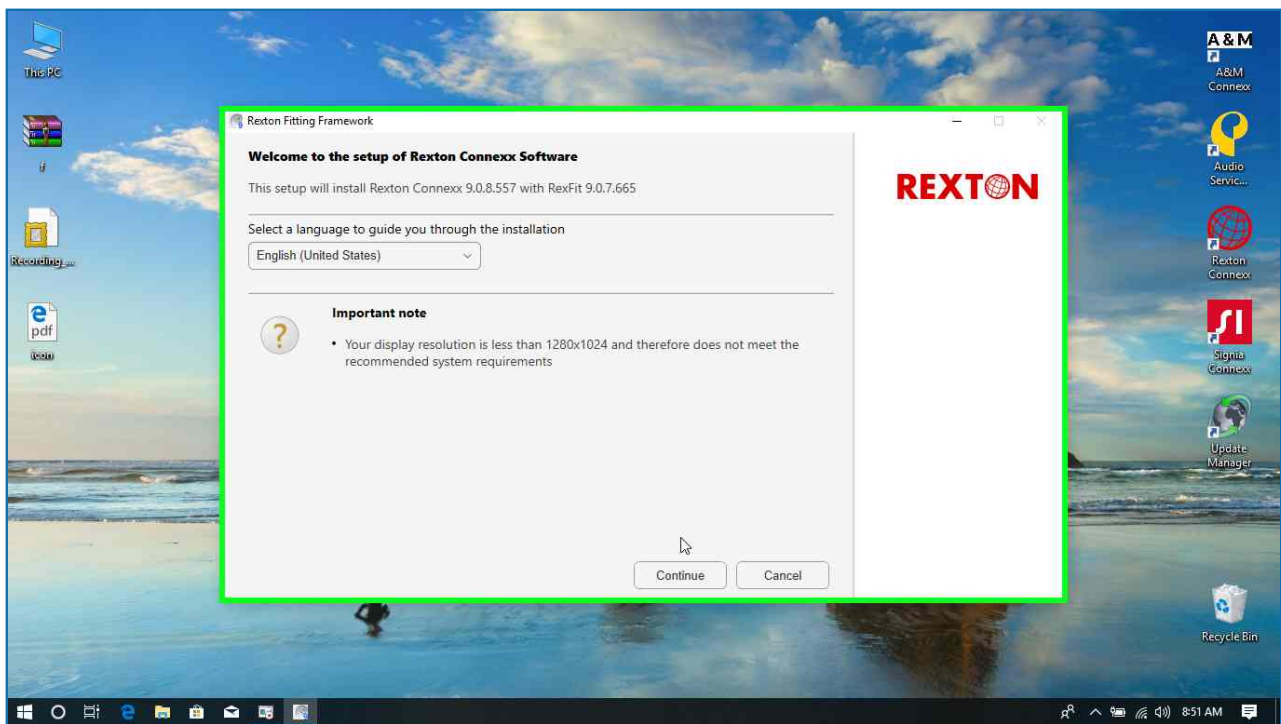


Step 16: (7/30/2019 8:51:52 AM) User left click on "Continue (text)" in "Rex"

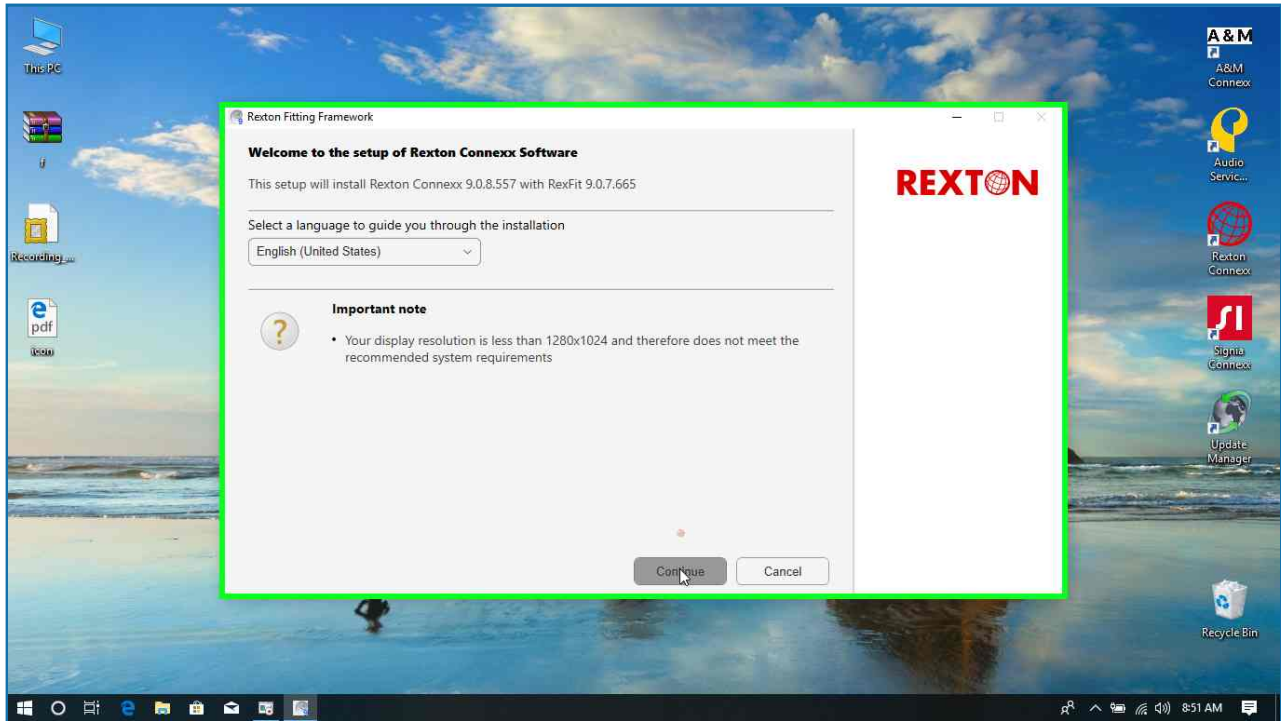


Step 17: (7/30/2019 8:51:54 AM) User left click on "Rexton Fitting Framework (window)" in "Rex"

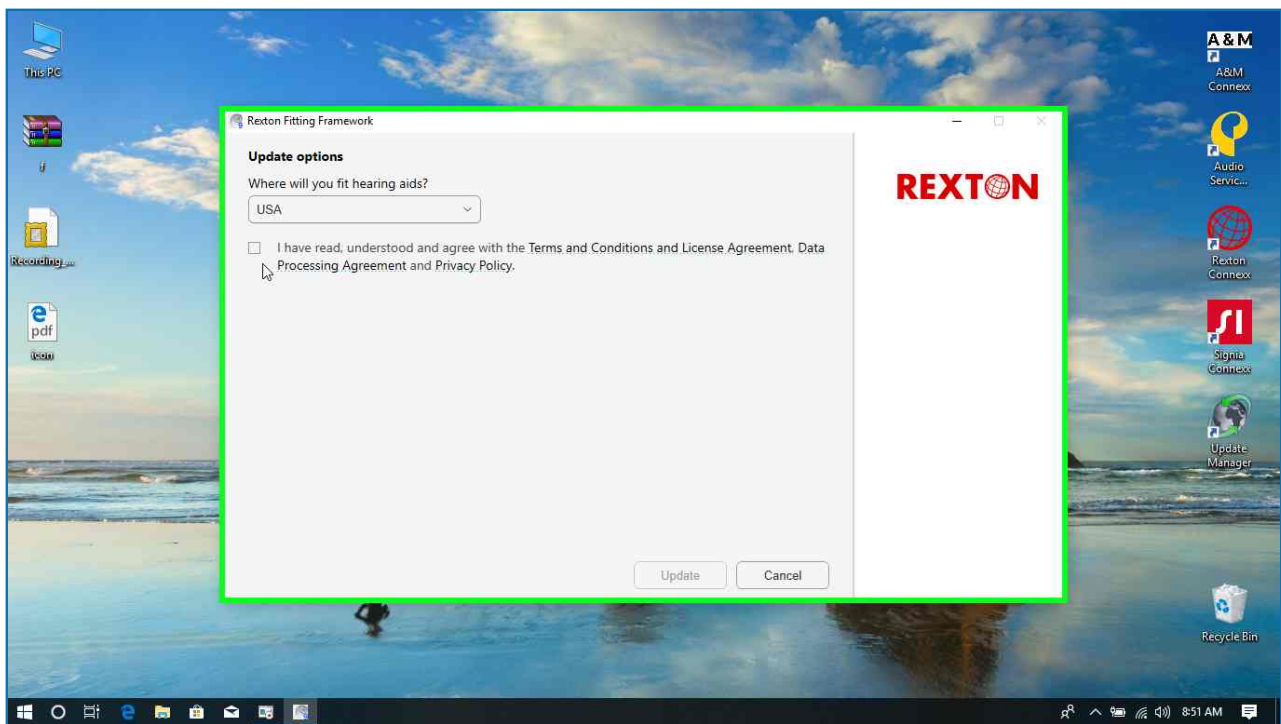


Step 18: (7/30/2019 8:51:55 AM) User left click on " (text)" in "Rex"

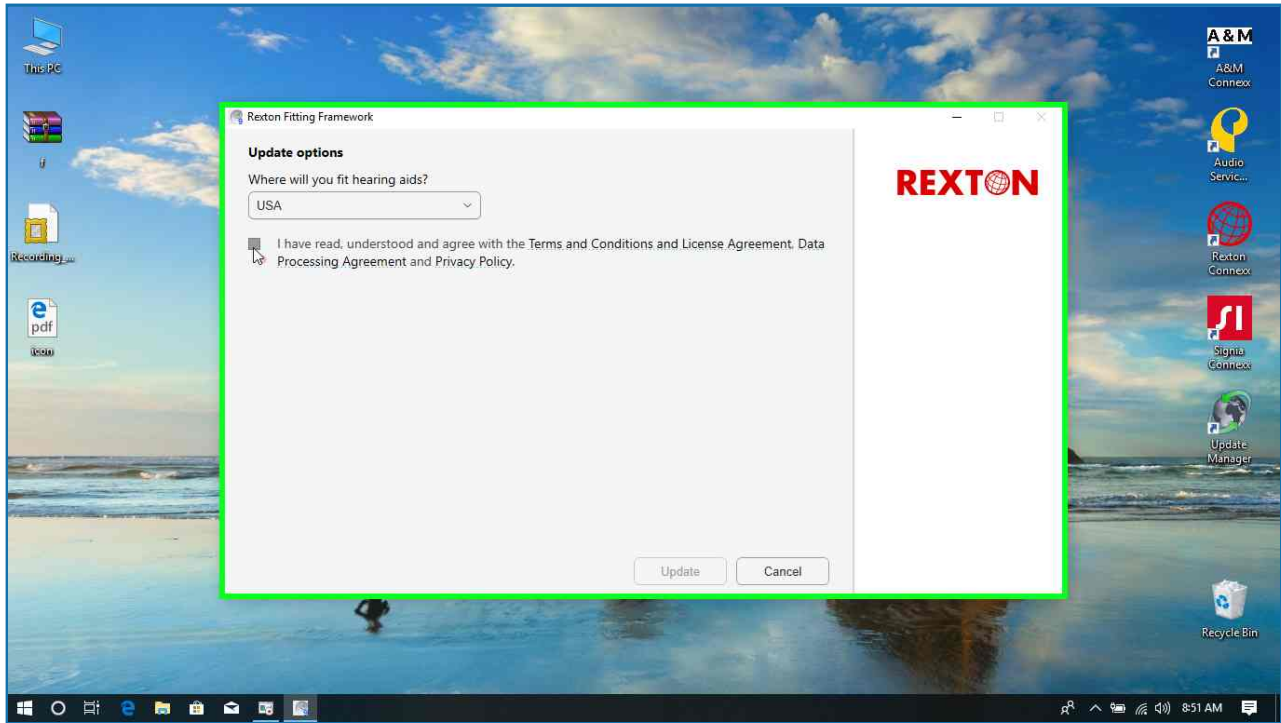


Step 19: (7/30/2019 8:51:57 AM) User left click on "Update (text)" in "Rex"

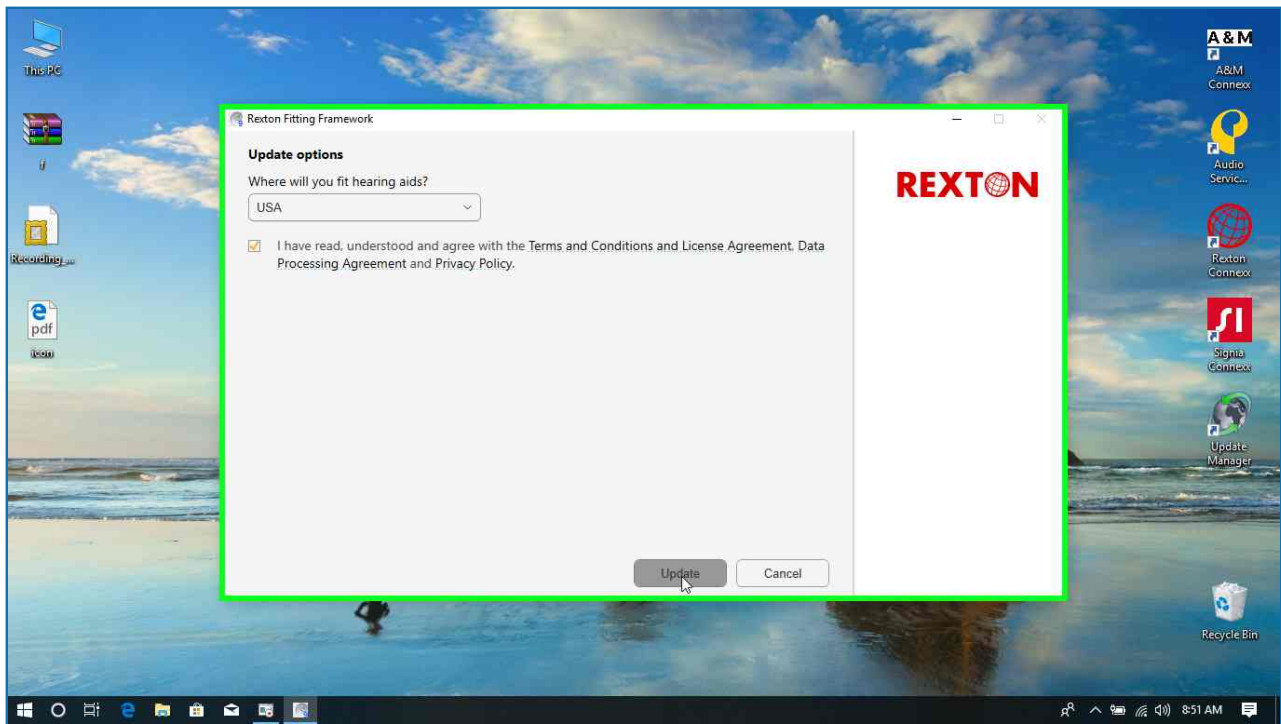

Step 20: (7/30/2019 8:52:00 AM) User left click on "SHS.SAT.Bootstrapper.CustomBA.Models.InstallationProgressModel (list item)" in "Rex"

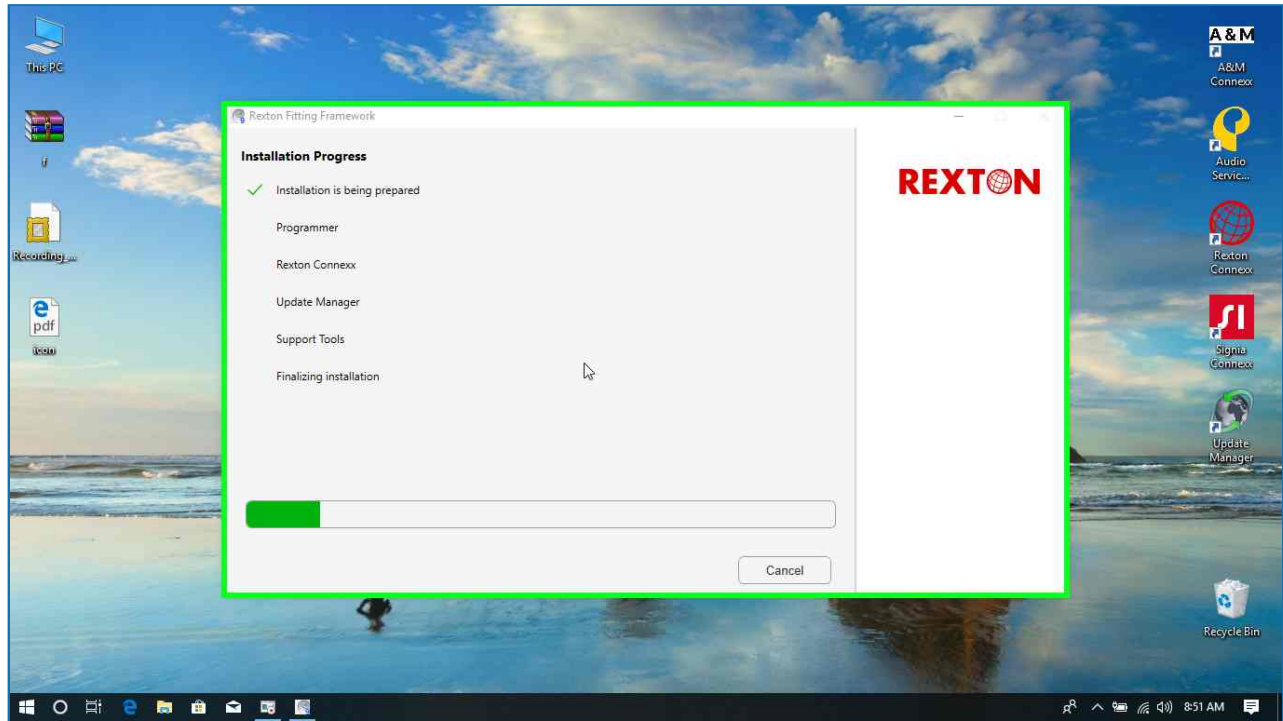


Step 21: (7/30/2019 8:54:44 AM) User left click on "SHS.SAT.Bootstrapper.CustomBA.Models.InstallationProgressModel (list item)" in "Rex"

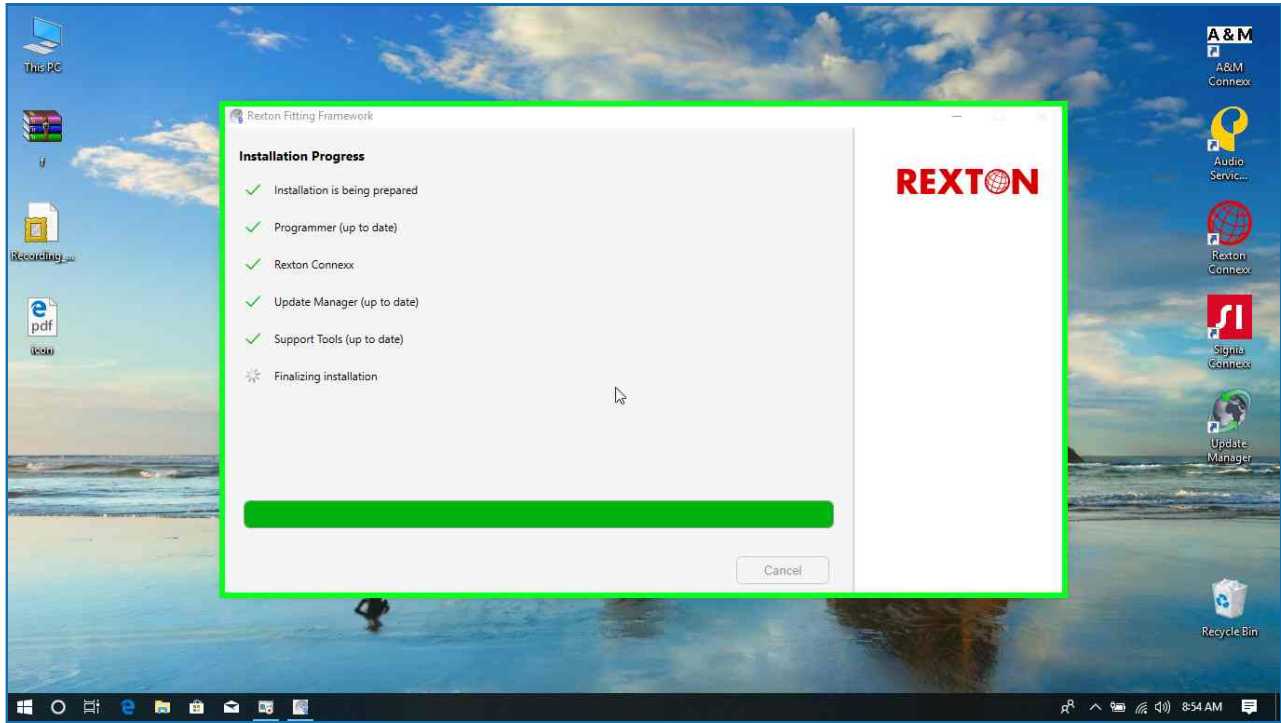


[Previous](#) [Next](#)

Step 22: (7/30/2019 8:55:09 AM) User left click on "Rexton Fitting Framework (window)" in "Rex"

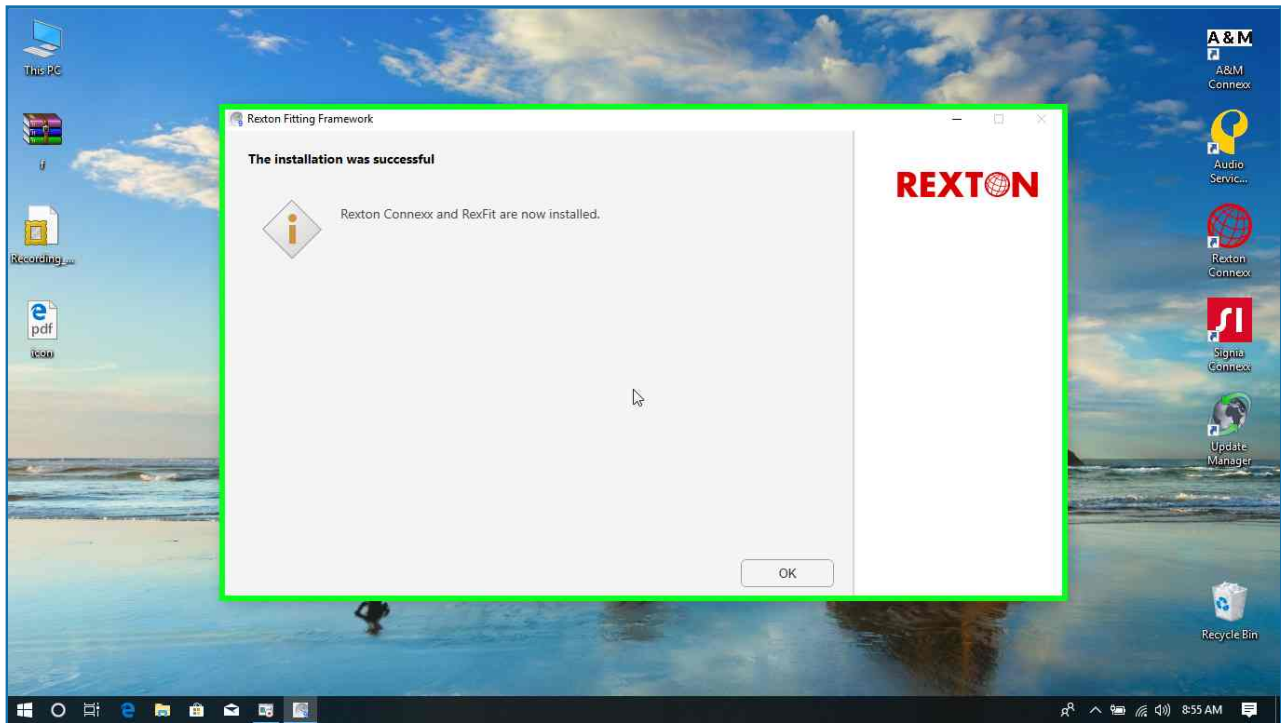

[Previous](#) [Next](#)

Step 23: (7/30/2019 8:55:10 AM) User left click on "OK (button)" in "Rex"

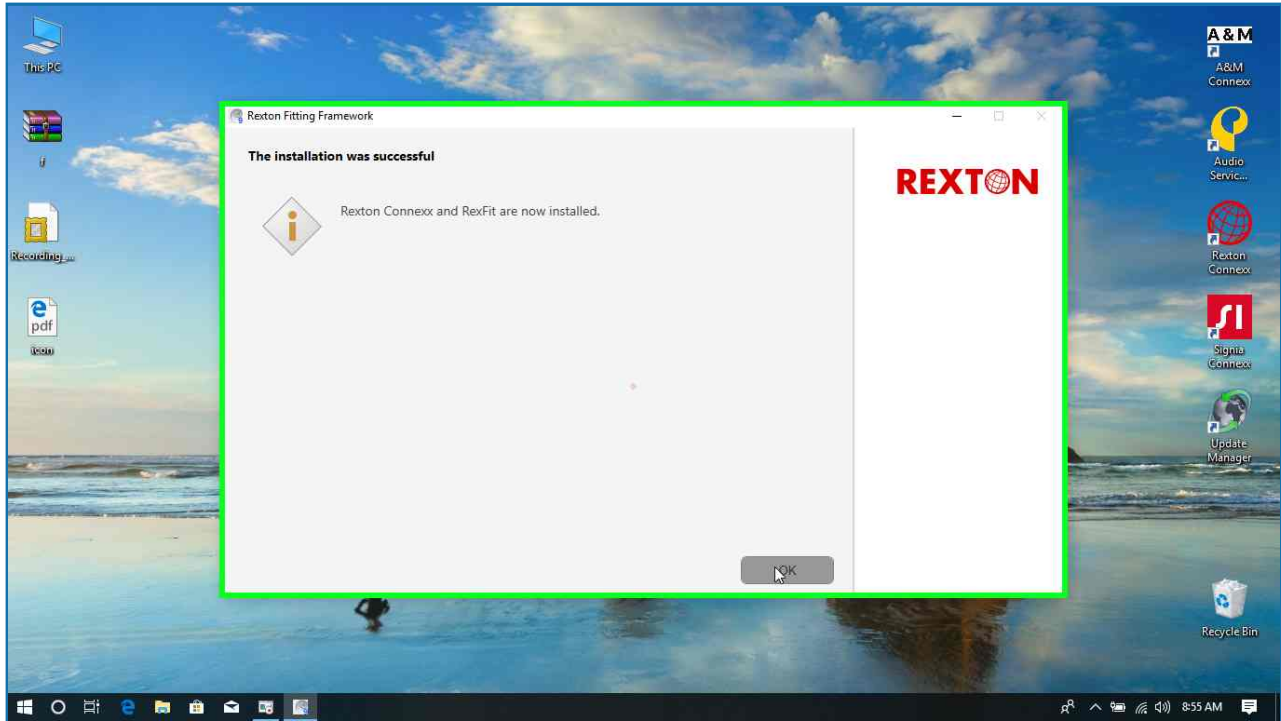

[Previous](#) [Next](#)

Step 24: (7/30/2019 8:55:52 AM) User left click on "Rexton Connexx (list item)" in "Pro"



Step 25: (7/30/2019 8:55:53 AM) User left double click on "Rexton Connexx (list item)" in "Pro"



Step 26: (7/30/2019 8:56:23 AM) User left click on "9.0.8.557 (text)" in "Spl"

